

Before Inhalation

● Using an autoclave

Refer to the sterilizer's instruction manual and precautionary information.

Parts should be placed in the autoclave for 15 minutes at a temperature of 121 °C.

Notes:

- When sterilizing in an autoclave, exercise care so that no parts come into contact with metal parts that are in contact with the sterilizer heater or heater. Because the heater becomes hot, any parts that come into contact with it may melt or become misshapen.
- Some parts may become slightly discoloured over time, but this does not indicate a problem with their material properties.


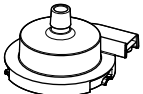

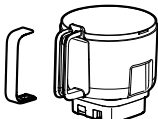
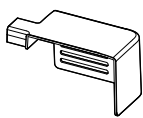
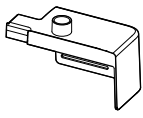
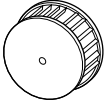
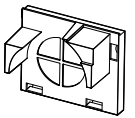
● Disinfection chart

Parts can be disinfected using the methods listed in the chart below. For more information about disinfecting the main unit, see "Disinfecting the main unit". (👉 Page 7) For more information about the disinfection process, see the disinfectant's instructions and precautions.

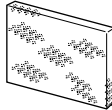
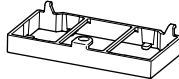

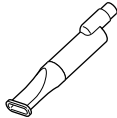



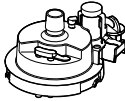
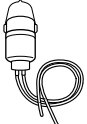
Note: Materials are indicated by the following abbreviations:

Acrylonitrile butadiene styrene resin: ABS, Poly phenylene sulfide resin: PPS, Polyethylene resin: PE, Polycarbonate resin: PC, Polymethylpentene: PMP, Polypropylene resin: PP, Polystyrene resin: PS, Polyvinyl chloride resin: PVC, Thermoplastic elastomer: TPE

○ : applicable ✕ : not applicable

Parts	Materials	Boiling water	Chemical disinfectant	Autoclave
Medication cup 	PP	✕	○	✕
Medication cup cover (with silicon packing (Washer)) 	Medication cup cover: PMP Silicon packing (Washer): Silicon rubber	○	○	○
Medication cup holder 	PP	✕	○	✕
Removable water tank Note: Disassemble the handle cover before disinfection. 	Removable water tank: ABS/PPS Handle cover: ABS	✕	○	✕
Fan cover 	ABS	✕	○	✕
Fan cover for bacterial filter 	ABS	✕	○	✕
Fan 	PP	✕	○	✕
Air filter case 	PP	✕	○	✕

Before Inhalation

Parts	Materials	Boiling water	Chemical disinfectant	Autoclave
Air filter 	PS	×	×	×
Protective Filter Case 	ABS	×	○	×
Protective Filter 	PS	×	×	×
Mouthpiece 	PP*	○	○	○
Inhalation mask (L) set 	Mask: TPE String: Rubber**	○ (Do not disinfect string.)	○ (Do not disinfect string.)	○ (Do not disinfect string.)
Inhalation mask (S) set 	Mask: TPE String: Rubber**	○ (Do not disinfect string.)	○ (Do not disinfect string.)	○ (Do not disinfect string.)
Inhalation hose M Inhalation hose L 	Hose: TPE Hose ends: TPE	○	○	○
Long time Nebulization set 	Cover: PMP Cap: Silicon rubber Silicon packing (Washer): Silicon rubber	○	○	○
Supply Medication set 	Bottle: PP Silicon packing (Washer): Silicon rubber Tube: Silicon rubber Hanger: Stainless steel	×	○ (Do not disinfect hanger.)	×

*Heat-resistant polypropylene

**The band is not made with natural rubber latex.